

# NIPPY Clearway

## Clinicians Quick Set-up Guide

### About The Device

- The NIPPY Clearway is a mechanical insufflation-exsufflation device (MI-E).
- The NIPPY Clearway is licensed as a therapy device and can be used for adults and paediatric patients as directed by a competent physician.
- The NIPPY Clearway can be used with a face mask, via a mouthpiece or connected directly to the tracheostomy tube.

### Un-locking/Locking the device

- To unlock the device, push and hold simultaneously the + and – button for 3 to 4 seconds until the padlock on the bottom right of the screen unlocks.
- To lock the device, again, hold the + and – button simultaneously for 3 to 4 seconds until the padlock on the screen shows as locked.

### Clear Settings

- To clear the settings, press the menu button and then select option 4 – clear patient settings. Then follow the instructions on the device.

### Set Up - Manual Mode

- Once the settings are cleared you will be in manual mode as default.
- To set the insufflation pressure, press the insufflation button, then use the + and – keys to select the required pressure, confirm by pressing set.
- To set the exsufflation pressure, press the exsufflation button, then use + and – keys to select the required pressure, confirm by pressing set.
- Using either the manual control or front button, you can deliver the therapy manually. You control the insufflation and exsufflation times as well as the pause time manually.

### Set Up – Basic Auto

- To select basic auto mode, press the mode button, use the – key to scroll down to basic auto mode, number 2 in the menu, and press set to select.
- To set the insufflation and exsufflation pressures, press the insufflation or exsufflation button, then use the + and – keys to select the required pressure, confirm by pressing set.
- To set the insufflation (Ti) and exsufflation times (Te), select the appropriate button, Ti or Te and use the + or – keys to set the time, confirm the correct time by pressing set.
- To set the pause, press the pause button, then using the + or – keys select the appropriate pause time, confirm this by pressing set.
- To start the therapy press the start button, this will give maximum 20 cycles, to stop the therapy before this press the start button again.

**For further training or to obtain the NIPPY Clearway clinical guidelines please contact your local representative**

# NIPPY Clearway

## Consumables

NIPPY Clearway Circuit		0960.101
Full Face Mask		0960.106 /? I = Infant P = Paediatric S = Small M = Medium L = Large XL = Extra Large
Mouthpiece & Nasal Clip		0960.105
Flexible Catheter Mount		0960.102
Anti-Bacterial Filter		0960.107
Oxygen Coupler		0960.108
Air Inlet Filter		0960.116
Trolley		0960.115
External Battery		0960.111
External Battery Charger		0960.112
Manual Control		0960.109
Mobility Bag		0960.114